
Gentler

THE GENTLE WAY FOR SMART FARMING

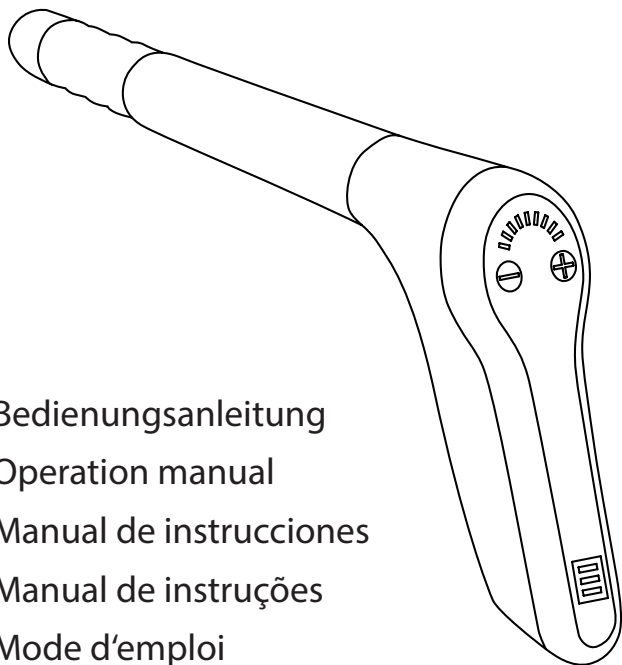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Operation manual

Manual de instrucciones

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Mode d'emploi

Sprachen

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Content

Introduction	11
Uses	11
Package contents	11
Description of components	11
Operation	13
Technical specifications	15
Care	16
Storage	16
Disposal	16
Troubleshooting	16
Safety Instructions	17
Guarantee	17

Introduction

The GENTLER is a veterinary product used to immobilize animals. It ensures the careful immobilization of the animal during animal husbandry activities (such as claw care, branding, etc.) or veterinary examinations.

The technology of the GENTLERS sends an electronic pulse, which naturally stimulates the nervous system of the animal. The animal remains motionless for the duration of use.

Uses

For example, for medical examinations / incision / ring banding / ear markings / vaccinations / brand markings / claw care / tagging / clipping and trimming

Package contents

- [1] - GENTLER
- [2] - Power adaptor
- [3] - Operating instructions



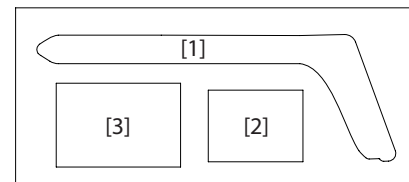
Fig.1

Description of components

- [4] - ergonomically shaped tip
- [5] - optimized insulation surface for reduction of the contact points in the animal
- [6] - Powerful industry standard Li-Ion battery
- [7] - Easy charging of the battery using a magnetically centering plug
- [8] - Displays the current intensity
- [9] - Plus button to increase the current intensity
- [10] - Minus button to reduce the current intensity
- [11] - Battery charge indicator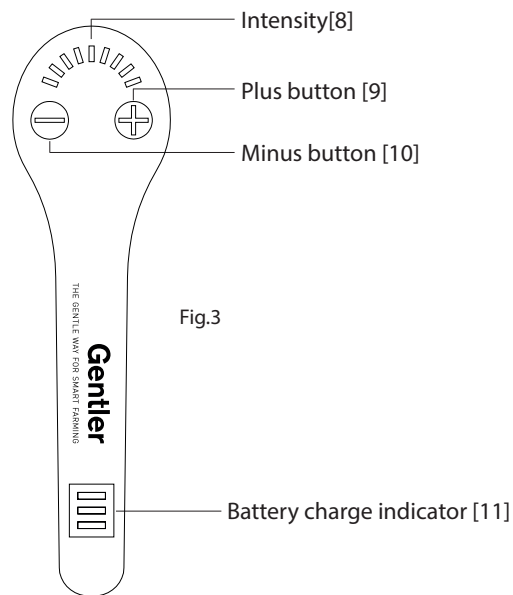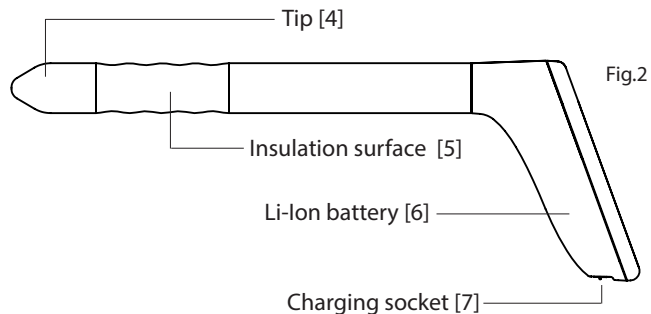

Operation

During use, the animal should be able to lean sideways, so it can maintain its reduced balance during immobilization.

Important: If the animal is kept still by means of a head clamp, care must be taken to ensure that it has sufficient freedom to move around during the immobilization. An overly narrow head clamp during application can lead to interruption of blood flow to the head and thus cause considerable physical damage to the animal.

The device should be switched off whenever possible before being inserted into the animal as well as before being removed from the animal.

First raise the tail of the animal and insert the probe of the **switched off** device into the rectum. Place the tail to one side over the protruding part of the handle.

Press and hold the plus button [9] for 2 seconds to switch on the device.

The animal will now lift its leg on the side to which the tail is aligned, and will remain motionless.

The intensity of the current pulse [8] is initially 50% after switching on.

If the output intensity of the current pulse is not sufficient or too high, increase or decrease the intensity by pressing the plus button or minus button.

The initial value can also be adjusted individually. To do this, select the desired value and then hold down both the plus button [9] and the minus button [10] for 3 seconds or longer. This will save the value and recall it the next time the Gentler is turned on.

The Gentler has a time-controlled automatic shut-off which automatically turns off the device after 1 hour of inactivity; all the LEDs flash at short intervals 1 minute prior to the automatic shut-off. If you press a key during this time, the automatic shut-off is delayed by one more hour.

Begin the treatment or grooming.

To terminate the immobilization, press and hold the minus button [10] for 2 seconds to turn off the Gentler. Now pull the probe out of the rectum of the animal. The animal will return to its normal state after the Gentler is switched off.

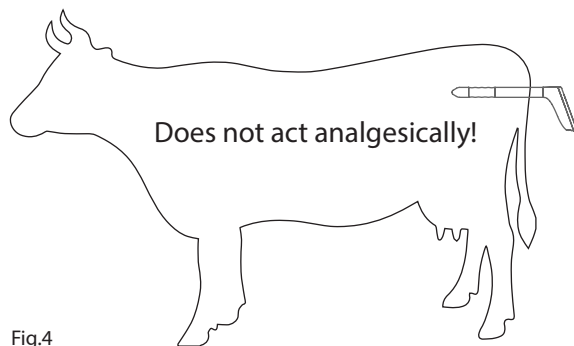
Fig.4

Not a substitute for anesthetics!!!

It is the responsibility of the user to utilize the GENTLER application purely for the immobilization of the animal. The device must not be used as a substitute for anesthetics during, for example, surgical procedures. **It does not therefore replace the use of conventional anesthetics.**

The device has one (1) pcs. Industry quality battery.

The operating time of the fully charged batteries is approximately 8 hours (depending on the duration of use). The battery charge status indicator [11] has 3 LED diodes which signal different states.

1. All 3 LED diodes lit green:

Battery is fully charged.

2. The lower 2 LED diodes lit green:

Battery is half full.

3. The bottom LED lit green:

Battery is charged to one third.

4. The bottom LED is red:

Battery is not charged and the device must not be used.

Regular rechargeable battery recharging is required after 2 months at the latest.

Some days before use, dry feed is to be avoided during the feeding of the animals, since this can lead to a dry bowel and reduced conductivity of the electrical signal at the treatment site.

Technical specifications

Length:	401,4 mm
Height:	141 mm
Width (Grip):	51,15 mm
Diameter (Electrode):	32 mm
Weight:	390 Gram
Battery type:	2500 mAh, 3,7V
Input voltage:	9 - 12V DC
Input current:	2A
Output voltage:	5-17V @32Hz

Care

Thoroughly clean the appliance after use in order to avoid corrosion by animal acids.

IMPORTANT - In case of occasional use, the battery should be recharged at least every 2 months.

Operation at low temperatures (< 0°C) may result in reduced operating time.

Do not expose the device to direct sunlight for long time. The associated elevated temperatures may damage the device.

Avoid prolonged storage of the device at temperatures over 40°C. This significantly reduces the life of the built-in batteries.

Storage

Please store the product in a closed, dry place. Prevent unauthorized use. Faulty operation can lead to physical damage.

Disposal

In the case of a total damage, please ensure proper disposal according to applicable national law. We recommend that an inspection be performed by an authorized specialist.

Troubleshooting

Some days before use, dry feed is to be avoided during the feeding of the animals, since this can lead to a dry bowel and reduced conductivity of the electrical signal at the treatment site.

Safety Instructions

Ensure that the product is serviced by trained personnel only.

Only carry out repairs to the GENTLER exclusively by our authorized specialist dealers. Improper repairs may result in considerable danger for the user.

Improper repairs will also result in the cancellation of your warranty claim - in particular, opening of the device by unauthorized persons.

Protect the equipment from unauthorized use by third parties such as children or adults who have not been instructed in the use of the Gentler.

Guarantee

There is a two-year worldwide manufacturer's warranty from the date of purchase. The sole contact person for the warranty is the seller or distributor in the respective country.

The guarantor is KN Sales.

K+N Sales UG, Lessingstraße 21, 64407 Fränkisch-Crumbach;
Tel.: +49 6161 / 877 169 83; E-Mail: info@knsales.net.

It applies throughout Europe. The statutory warranty (liability for defects) is not restricted by the guarantee. The warranty covers the assumption of all repair and shipping costs for defects occurring during the warranty period and which are not based on wear, normal consumption or improper handling. In case of guarantee claim, please contact the aforementioned guarantor. After sending the product and submitting the proof of purchase, the product will be repaired free of charge or replaced by an equivalent product.

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